m213A

AEROSOL A PISTONE

PISTON TYPE COMPRESSOR NEBULIZER

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medicura®

GUARANTEED
2
YEARS

ISTRUZIONI D'USO

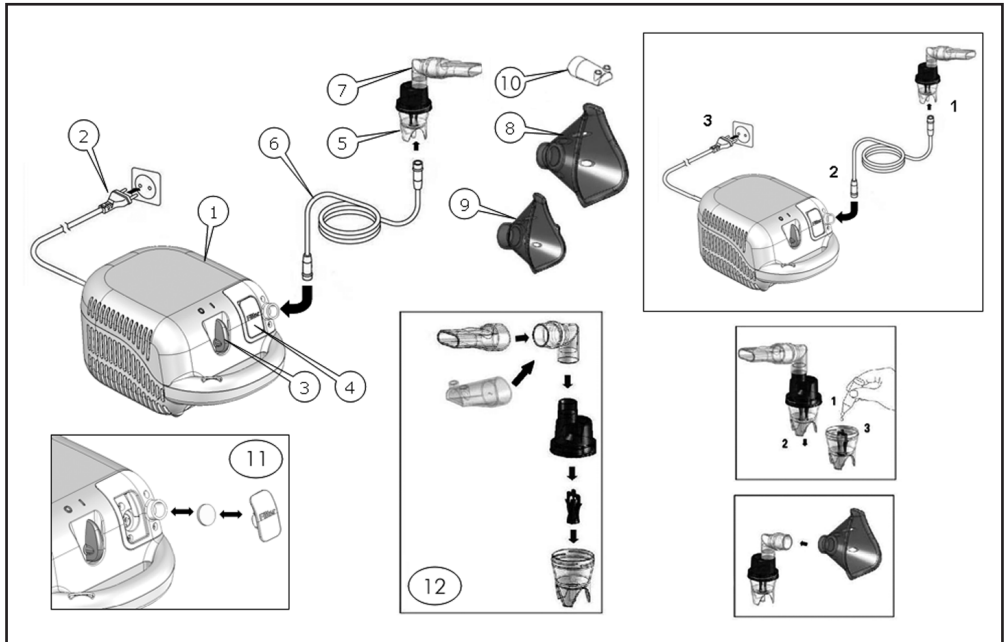
INSTRUCTION MANUAL

MANUEL D'INSTRUCTIONS

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MANUAL DE INSTRUCCIONES

medicura®

SIMBOLOGIA - SYMBOLS - SYMBOLIQUE - VERWENDETE SYMBOLE - SIMBOLGÍA

- Apparecchio di classe II/Class II/Appareil de classe II/Schutzklasse II/Aparato de clase II.
- Apparecchio di tipo BF/Type BF device/Appareil de type BF/Geräteyp BF/Aparato de tipo BF.
- Interruttore spento/Switch "OFF"/Interrupteur éteint/Ein-Ausschalter ausgeschaltet/Interruptor apagado.
- Interruttore acceso/Switch "ON"/Interrupteur allumé/Ein-Ausschalter eingeschaltet/Interruptor encendido.
- Corrente alternata/Alternating current/Courant alternatif/Wechselstrom/Corriente alterna.
- Non utilizzare l'apparecchio mentre si fa il bagno o la doccia/Never use the unit while having a bath or shower/Ne pas utiliser l'appareil pendant le bain ou la douche/Das Gerät darf nicht während des Badens oder in der Dusche/No utilizar el aparato mientras se hace el baño o la ducha.
- È obbligatorio leggere attentamente le istruzioni prima di utilizzare questo dispositivo/It is compulsory to carefully read the instructions before using this device/Il est obligatoire de lire attentivement les instructions avant d'utiliser ce dispositif/Vor der Verwendung des Gerätes ist die Bedienungsanleitung aufmerksam zu lesen/Es obligatorio leer atentamente las instrucciones antes de usar este dispositivo.
- Tenere all'asciutto/Keep dry/Garder au sec/Trocken halten/Mantener seco.

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PISTON TYPE COMPRESSOR NEBULIZER

- | | | | |
|----------------------------|---------------------------------|--------------------------|------------------------------------|
| 1 Piston compressor | 4 Air filter compartment | 7 Mouthpiece | 10 Nosepiece |
| 2 Power cord | 5 Nebjet Nebulizer | 8 Adult face mask | 11 Replacing air filter |
| 3 ON/OFF Switch | 6 Air hose | 9 Child face mask | 12 Assembling nebuliser kit |

Dear Customer,

this nebuliser is a high quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. This medical device generates aerosols with variable characteristics to adapt to the patient's respiratory pathology and generating a medicinal mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your **m213A** product. For any further questions you may have, please ask your chemist or usual retailer.

IMPORTANT WARNINGS

This device may only be used for the purposes described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application. This device is not suitable for anaesthesia and lung ventilation. This device should only be used with original accessories as shown in these instructions. Do not use the device if you think it is damaged or notice anything unusual. Never open the device. This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section. Protect it from: - water and moisture - extreme temperatures - impact and dropping - contamination and dust - direct sunlight - heat and cold. Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.

All accessories should only be used by one person. Never bend the nebuliser over 60°.

- The unit should not be used in the presence of anesthetic mixture inflammable with air, oxygen or nitrous oxide.
- Device not protected against splashes.
- Keep the device away from splashes and water jets. In case of transportation or in the presence of possible water drips, use the bag provided.
- In view of their length, the power cord and connection hose could constitute a strangulation hazard.

 Ensure that children do not use the device unsupervised some parts are small enough to be swallowed.

 Use of this device is not intended as a substitute for consultation with your physician.

PREPARATION AND USAGE OF THIS DEVICE

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- 1.** Assemble the nebuliser kit **12**. Ensure that all parts are complete.
- 2.** Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- 3.** Connect the nebuliser with the air hose **6** to the compressor **1** and plug the power cord **2** into the socket.
- 4.** Switch the ON/OFF switch **3** to position «**I**» to turn on the device and place the mouthpiece **7** in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between mouthpiece **7**, nosepiece **10**, adult **8** or child face mask **9** and make sure that it encloses the mouth and nose area completely.
- 5.** Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
- 6.** After completing the inhalation period recommended by your doctor, switch the ON/OFF switch **3** to position «**0**» to turn off the device and unplug it from the socket.
- 7.** Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

CLEANING AND DISINFECTING

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor **1** and the air hose **6** should be cleaned with a clean, moist cloth.

-  Always wash your hands well before cleaning and disinfecting the accessories.
-  Do not expose the compressor to water or heat.
-  Replace the air hose for each treatment with a new patient or in case of impurities.

 **Unplug the device before cleaning.**

Cleaning with water

Wash all nebuliser components (except air hose) under warm tap water (max 60 °C) for about 5 minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer. Rinse thoroughly making sure that all deposits are washed away and leave to dry.

Disinfecting

All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer. Disinfectants are usually available at pharmacies.

Sterilizing with steam

All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). After sterilization always let all components cool down to ambient temperature before further use.
Do not repeat sterilization cycle when components are still warm.

MAINTENANCE, CARE, AND SERVICE

Order all spare parts from your dealer or pharmacist.

It is advisable replace nebulizer after some 100 to 120 treatments on single patient or after about 20 sterilization cycles. Check the filter continually for cleanliness and replace it if dirty, or after a maximum of 3 months use. Spare filters are provided with the device. To replace the filter, open the air filter compartment **4** marked «**Filter**» and change the filter **11**.

MALFUNCTIONS AND ACTIONS TO TAKE

The device cannot be switched on

Ensure the power cord **2** is correctly plugged into the socket. Ensure the ON/OFF switch **3** is in the position «**1**».

The nebuliser functions poorly or not at all

Ensure the air hose **6** is correctly connected at both ends. Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one. Ensure the nebuliser **5** is correctly assembled **12**. Ensure the required medication has been added.

TECHNICAL DATA

Piston electrocompressor with thermal cut

Nominal voltage: see data label

Dimensions: 130x190x100 mm

Weight: 1.1 Kg approx.

Temporary use: 20/40 min. ON/OFF

MDD 93/42/EEC - Class of risk IIa

Maximum pressure: 185 kPa

Air flow: 10 Lt/min

Noise level: 58 dBA (EN13544-1)

Neb rate: 0.35 ml/min (with 0.9% NaCl solution)

Nebulizer capacity: min. 2 ml - max. 12 ml

Operating pressure: 60 kPa (0.6bar)

Operating flow: 5 l/min

MMAD: 2.61 µm GSD: 2.75 Aerosol Output: 143 µl Output Rate: 87 µl/min

Expected service life: 5 years

Operating Temperature/Humidity: +10° C to +40° C/10% to 95% RH

Storage and Transport Temperature/Humidity/Air Pressure: -25° C to +70° C /10% to 95% RH /690 - 1060hPa 

IMPORTANT INFORMATION REGARDING ELECTRO MAGNETIC COMPATIBILITY (EMC)

Electronic devices (PC's, mobile phones, televisions, etc) that generate electromagnetic interference can affect the correct functioning of medical devices. This situation can lead to incorrect functioning creating unsafe conditions. In order to show compliance with the Directives on Electromagnetic Compatibility (EMC), all the tests introduced by the standard EN60601-1-2:2015 have been implemented. This standard defines the levels of immunity to electromagnetic interference as well as maximum levels of electromagnetic emissions for medical devices: **3A HEALTH CARE** products comply with the levels specified in the standard. However, when the device is in use, it is best practise not to use mobile phones or other electronic devices which generate strong electromagnetic fields near the medical device. This may create dangerous conditions that put the successful outcome of treatment at risk. On the basis of experience and tests conducted, it is recommended that a minimum distance of 7 m be kept: if the distance is less, the correct functioning of the device should be verified. Documentation regarding the standard EN60601-1-2:2015 is available from **3A HEALTH CARE** (see the address in the instruction manual) and on the website www.3-a.it.

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TERMS AND CONDITIONS:

The guarantee is valid for 24 months.

This guarantee applies only if it has been duly filled in and is submitted with the receipt showing the date of purchase.

The guarantee covers the replacement or repair of parts making up the appliance which were faulty at source due to manufacturing faults. After the guarantee has expired, the appliance will be repaired against payment.

The manufacturer declines any responsibility for damage to persons, animals or property due to misuse of the appliance and failure to observe the directions contained in the instructions.

LIMITS:

All rights under this guarantee and any responsibility on our part will be voided if the appliance has been:

- mishandled by unauthorized persons;
- improperly used, stored or transported.

The guarantee does not cover damage to the outward appearance or any other that does not prevent regular operation.

If any faults should be found despite the care taken in selecting the materials and in creating the product, or if any information or advice are required, please contact your local dealer.

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DISPOSAL PROCEDURE (Dir. 2012/19/Ue-WEEE) The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. **Caution!** The wrong disposal of electric and electronic equipment may involve sanctions.

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POLY POOL S.p.A.